**THUASNE**

# Dynastab® Dual

<b>fr</b>	Attelle d'immobilisation du poignet.....	5
<b>en</b>	Wrist immobilisation splint.....	6
<b>de</b>	Handgelenkorthese zur Immobilisation.....	7
<b>nl</b>	Immobilisatiebrace voor de pols.....	9
<b>it</b>	Tutore di immobilizzazione per il polso.....	10
<b>es</b>	Férula de inmovilización de la muñeca.....	11
<b>pt</b>	Tala de imobilização do punho.....	12
<b>da</b>	Immobiliserende håndledsskinne.....	14
<b>fi</b>	Immobilisoiva rannetuki.....	15
<b>sv</b>	Immobiliseringsskena för handleden.....	16
<b>el</b>	Νάρθηκας ακινητοποίησης καρπού.....	17
<b>cs</b>	Zápěstní imobilizační ortéza.....	19
<b>pl</b>	Orteza do unieruchomienia nadgarstka.....	20
<b>lv</b>	Plaukstas locītavas imobilizācijas saite.....	21
<b>lt</b>	Riešo imobilizavimo įtvaras.....	23
<b>et</b>	Randme fikseerimise lahas.....	24
<b>sl</b>	Opornica za imobilizacijo zapestja.....	25
<b>sk</b>	Imobilizačná dlaha na zápästie.....	26
<b>hu</b>	Csuklórögzítő.....	28
<b>bg</b>	Шина за имобилизация на китка.....	29
<b>ro</b>	Atelă de imobilizare a încheieturii mâinii.....	30
<b>ru</b>	Шина для имобилизации запястья.....	32
<b>hr</b>	Udlaga za imobilizaciju ručnog zgloba.....	33
<b>zh</b>	腕部固定夹板.....	34
<b>ar</b>	جيرة لتثبيت الرسغ.....	39

	1	13 - 16 cm
	2	16,5 - 19,5 cm
	3	20 - 23 cm

<b>fr</b>	Tour de poignet
<b>en</b>	Wrist circumference
<b>de</b>	Handgelenkumfang
<b>nl</b>	Polsomtrek
<b>it</b>	Circonferenza del polso
<b>es</b>	Contorno de la muñeca
<b>pt</b>	Perímetro do punho
<b>da</b>	Håndleddets omkreds
<b>fi</b>	Ranteen ympärymitta
<b>sv</b>	Handledens omkrets
<b>el</b>	Περίμετρος καρπού
<b>cs</b>	Obvod zápěstí
<b>pl</b>	Obwód nadgarstka
<b>lv</b>	Plaukstas locītavas apkārtmērs
<b>lt</b>	Riešo apimtis
<b>et</b>	Randme ümbermõõt
<b>sl</b>	Obseg zapestja
<b>sk</b>	Obvod zápästia
<b>hu</b>	Csukló kerülete
<b>bg</b>	Обиколка на китката
<b>ro</b>	Circumferința încheieturii
<b>ru</b>	Обхват запястья
<b>hr</b>	Obujam ručnog zgloba
<b>zh</b>	腕圍
<b>ar</b>	محيط الرسغ

<b>fr</b>	Immobilisation
<b>en</b>	Immobilisation
<b>de</b>	Ruhigstellung
<b>nl</b>	Immobilisatie
<b>it</b>	Immobilizzazione
<b>es</b>	Inmovilización
<b>pt</b>	Imobilização
<b>da</b>	Støtte
<b>fi</b>	Immobilisoi
<b>sv</b>	Immobilisering
<b>el</b>	Ακινητοποίηση
<b>cs</b>	Znehybnění
<b>pl</b>	Unieruchomienie
<b>lv</b>	Imobilizācija
<b>lt</b>	Imobilizavimas
<b>et</b>	Liikumatuks muutmine
<b>sl</b>	Imobilizacija
<b>sk</b>	Imobilizácia
<b>hu</b>	Rögzítés
<b>bg</b>	Обездвижаване
<b>ro</b>	Imobilizare
<b>ru</b>	Иммобилизация
<b>hr</b>	Imobilizacija
<b>zh</b>	固定
<b>ar</b>	التثبيت

**MD**

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## WRIST IMMOBILISATION SPLINT

#### Description/Destination

This device is a wrist splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

#### Composition

Polyamide – polyester – coton – elastane – polyurethane – polyoxyméthylène – aluminium.

#### Properties/Mode of action

Immobilisation of the wrist via a rigid, malleable and anatomical palmar stay.

Aerated lining.

Removable strap to adjust compression level.

Bilateral product.

#### Indications

Wrist sprain.

Wrist tendinopathies.

Rheumatoid arthritis.

#### Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

#### Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient. Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

#### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### Instructions for use/Application

1. Pull the orthosis over the wrist to be immobilised, leaving the straps on the outside, and find the two stays covers.
2. Slip the metal stay @ into the stay cover located on the inside of the hand (from the bottom). To do this, the inscription on the stay must be towards the elbow and facing you. Once on, it must mould to the shape of your palm.
3. Insert the plastic stay @ into the second stay cover.
4. Strap @ : Wrap around the hand and insert the strap into the tightening buckle. Adjust the strap using the self-fastening tab.
5. Strap @ : Position the strap on the wrist. Wrap around the wrist and insert the strap into the tightening buckle. Adjust the strap using the self-fastening tab.

#### Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label.

Remove stays before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

#### Storage

Store at room temperature, preferably in the original packaging.

#### Disposal

Dispose of in accordance with local regulations.

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2. Glissez la baleine métallique © dans le cache-baleine situé à l'intérieur de la main (par le bas). Pour cela, l'inscription placée sur la baleine doit être orientée vers le coude et face à vous. Une fois mise, celle-ci doit épouser la forme de la paume de votre main.
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros