THUASNE®

GenuExtrem

fr	Genouillère proprioceptive à baleines souples	5
en	Proprioceptive knee support with flexible stays	7
de	Propriozeptive Kniebandage mit weichen Schienen	9
nl	Proprioceptieve kniebrace met soepele baleinen	11
it	Ginocchiera proprioceettiva con stecche flessibili	14
es	Rodillera propioceptiva con ballenas flexibles	16
pt	Joelheira proprioceptiva de almofadas flexíveis	18
da	Proprioceptisk knæbind med smidige stivere	20
fi	Proprioseptiivinen polvituki, taipuisat pystytuet	22
sv	Proprioceptivt knästöd med flexibla skenor	24
el	Ιδιοδεκτική επιγονατίδα με εύκαμπτες μπανέλες	26
cs	Proprioceptivní kolenní ortéza s elastickými kosticemi	29
pl	Elastyczna proprioceptywna orteza stawu kolanowego z fiszbinami giętkimi	31
lv	Proprioceptīva ceļgala ortoze ar elastīgiem atbalsta elementiem	33
lt	Proprioceptija gerinantis kelio įtvargas su lanksčiomis plokštelėmis	35
et	Proprioseptiivne põlvetagid elastsete külgtugedega	37
sl	Proprioceptivna opornica s prožnimi rebri	39
sk	Proprioceptívna kolenná ortéza s pružnými výstužami	42
hu	Proprioceptív térdrögzítő rugalmas merevítővel	44
bg	Проприоцептивна скоба за коляното с гъвкави опори	46
ro	Genunchieră proprioceptivă cu balane flexibile	48
ru	Проприоцептивный наколенник с гибкими усиливающими элементами	51
hr	Proprioceptivni steznik za koljeno s mekanim bočnim umecima	53
zh	带有柔性支撑条的自体感护膝	55
ar	دعامة ركبّة متعلّقة باستقبال الحس المزودة بشرائح مرنة	59

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации -

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en

PROPRIOCEPTIVE KNEE SUPPORT WITH FLEXIBLE STAYS

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 6 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.
- flexible side stays guaranteeing good hold.

This device can be used as a proprioceptive support for rehabilitation.

Composition

Textile components: polyester - polyurethane - polyamide - elastane.

Rigid components: silicone - steel - polyethylene.

Properties/Mode of action

Elastic fabric (compression: class 3 - 20-36 mmHg / 27.1-48 hPa) ensuring good support and proprioception.

Indications

Knee pain and/or swelling management.

Joint instability/laxity.

Protection and prevention during sports activities.

Conservative treatment of mild knee ligament injuries.

Compression for scar/burns healing.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose.

To make it easier to put on, use the pull tabs at the top of the knee brace.

Care/Maintenance

Machine washable at 40°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive

products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2047901 (2023-10)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186